



FDA Approved Kerasave with Amphotericin B

Please review the instructions for use below:

- Amphotericin B tablet will be added to KeraSave cornea preservation medium prior to corneal tissue being placed into the preservative per surgeons documented request.
- Prior to transplantation, **rinse cornea with BSS**; this promotes removal of Amphotericin B from tissue. If you are using a TransplantREADY® preloaded DSAEK or DMEK **exchange the Kerasave + Amphotericin B preservative with BSS**.
- Do not freeze

Kerasave with Amphotericin B additive is FDA approved as a cornea preservative. Kerasave, made by Alchimia in Italy, is a corneal storage medium with an optional Amphotericin B tablet additive. The combined presence of an antifungal agent and antibiotics promotes the microbiological safety of the corneal tissue.



LIONS
WORLD VISION
INSTITUTE™

So the world can see.

Formerly Lions Eye Institute for Transplant & Research

1410 N 21st Street, Tampa, FL 33605-5313 | 813.289.1200 | LWVI.org

v.03.28.2023